



Basic Powder Guidance

Vita Lumin Shade Guide

	A1	A2	A3	A3.5	A4	B1	B2	B3	B4	C1	C2	C3	C4	D2	D3	D4
Dentin Liner	10	10	11	11	11	12	10	13	13	12	14	14	11	15	15	15
Dentin	A1	A2	A3	A3.5	A4	B1	B2	B3	B4	C1	C2	C3	C4	D2	D3	D4
Enamel	58	58	59	59	60	57	58,59	59	59	60	59	59	60	60	59	59
Neck Translucent*	52	52	53	53	53,55	52	52	52,54	53,54	51,52	52,53	53,54	52,55	52	53,54	52,54
Make In* (mix 1:1 w/Dentin)	61	AD1	63	63	65	AD1	AD1	63	65	61	61/63	61/63	61/OD44	61/63	61/OD44	61/OD44
SI*	HT51;02	02	02;04	04	04	HT51;02	02	02;04	04	02	02	02;04	04;06	02;04	02;04	04
Clears	CL-0, OT, & TI-4 are transparent powders that work well with most shades															

* HT, MI, SI, and TI powders are not shade specific - the materials indicated are merely suggestions of complimentary colors for the indicated shades. See "How to Apply the Unique Powders of the Creation Porcelain" system for more information.

Shoulder Porcelains

Shade	A1	A2	A3	A3.5	A4	B1	B2	B3	B4	C1	C2	C3	C4	D2	D3	D4
SP Blend	22	22	22;24	24;25	24;25	22	22	24;25	24	22;25	25;24	25	25	22;25	24;25	22;25
SP Blend Ratio*	100%	100%	1:1	1:1	2:1	100%	100%	1:1	100%	1:1	2:1	100%	100%	4:1	1:1	3:1

* Closer shade matching can be achieved by including In Nova powders.

